Lacrisun

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Flacone multidose
da 10 ml
senza conservanti

Lubrificant
eye drops
for tear film
dysfunction

10 ml dispenser,
preservatives
free - OSD
(Ophthalmic
Squeeze Dispenser)

CE 0373
MEDICAL DEVICE

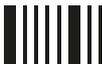
STERILE A

Rev. 0 dated 12/2019

IROMED
GROUP S.r.l.
Via Tempo del Cielo 3/5
00144 Rome (Italy)

Distribuito da
/ distributed by:
Diadema
Farmaceutici Srl
Via G. Malasoma,14/16
56121 PISA (Italy)

INDICATIONS

Lacrisun[®] is indicated for:

- Corneal cross-linking patients
- Patients affected by keratoconus
- Eye dryness
- Alteration in the continuity of ocular surface
- Post surgery hyperemia
- Foreign body and surgery trauma
- Alterations of ocular surface, related to metabolic disorders
- Environmental stress (pollution, smoke, air condition, etc.)
- Offence from harmful light wavelengths

INSTRUCTION FOR USE

- 1 - Remove the cap
- 2 - Gently squeeze the bottle and put 1 or 2 drops in each eye; repeat as required.
Immediately close the container after use: the solution can be used for up to 2 months after first opening.

COMPOSITION

Sodium Hyaluronate, Sodium Chloride, Sodium Phosphate monobasic monohydrate, Sodium Phosphate dibasic dedacahydrate, L-proline, L-glycine, L-lysine hydrochloride, L-leucine, Riboflavin Sodium Phosphate (0.075%), Vitamin E TPGS, water p.p.i. Preservatives free.

PRECAUTIONS

- The solution is for external ophthalmic use only.
- Store at temperature below 30°C
- Do not use in case of allergy or hypersensitivity to any of the components.
- Do not use in case of damaged bottle or if the seal is broken before first opening.
- Do not use after the expiry date shown on the package.
- During product application, do not touch the eye or any other surface with the dispenser tip.
- The expiry date refers to unopened and correctly stored product.
- Before use read carefully the present leaflet.
- Keep out of children's reach.
- Properly dispose after use
- Surface may stain surfaces or fabrics

STORAGE

Immediately close the container after use: the solution can be used for up to 2 months after first opening. Store away from heat sources and anyway at temperature below than 30°C.

VALIDITY

Do not use after the expiry date shown on the package, the expiry date refers to unopened and correctly stored product.

Keep the present leaflet together with product.

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ENG

Lacrisun® is an ophthalmic solution composed of Riboflavin/ Vitamin B2, Vitamin E, Hyaluronic acid salt and amino acids. Vitamin B2 and Vitamin E TPGS interact with the lipid layer of the tear film creating a protective coat to avoid hyper evaporation of tears. Sodium Hyaluronate mechanically interacts with the mucin layer of tear film binding water and stabilizing ocular surface for a prolonged period. The synergistic action of amino acids nourishes the ocular surface whilst regulating pH and osmolarity and protects stressed epithelium. Vitamin E TPGS also increases wettability of the ocular surface. The ophthalmic solution includes, in a balanced way, all the nutrients needed to promote recovery of optimal conditions of the ocular surface after surgical procedures.

Lacrisun® can be used after UV-A corneal cross-linking and during sun exposure to maintain hydration of the cornea.

Lacrisun® can be used whilst wearing contact lenses.

>>